Health o meter[®]

P r o f e s s i o n a l

Model 2210KL / 2210KL4 High Resolution Digital Neonatal / Pediatric Tray Scale

2210KL

2210KL4

User Instructions

PELSTAR, LLC 9500 West 55th St. McCook, IL 60525-7110 USA

www.hom scales.com

© Pelstar, LLC 2019

weigh**easier**[®]

Thank you for purchasing this Health o meter® Professional product. Please read this manual carefully, and keep it for easy reference or training.

TABLE OF CONTENTS

Cautions and Warnings.....	3
Specifications for This Scale	4
Certifications / Disposal	5
Assembly Instructions	6
Set Up Instructions.....	8
Operating Instructions.....	11
Calibration.....	14
Troubleshooting	15
Maintenance	16
Warranty	17

Note: This scale has been factory calibrated, and does not require calibration prior to use.

Please register your scale for warranty coverage at:

www.homscales.com/product-support/technical-support/product-warranty-registration

For User Instructions updates and revisions please go to:

www.homscales.com

CAUTIONS AND WARNINGS

INTENDED USE

This Health o meter® Professional neonatal / pediatric scale is intended to be used in a professional medical environment by trained medical staff. This product was designed to weigh pediatric patients who are safely positioned on the scale tray. Do not modify the product or use it for anything other than its intended purpose.

To prevent patient / caregiver injury or damage to your scale, please follow the instructions in this user manual very carefully.

- Do not exceed the weight capacity specified for this scale.
- For accurate weighing, this scale must be placed on a flat, stable surface.
- For accurate weighing, verify before each use the proper operation according to the procedure described in this manual.
- Do not use in the presence of flammable or explosive materials.
- If the scale becomes damaged, it should not be operated until properly serviced.
- Do not transport the scale with a patient or object on the scale.
- Ensure that the scale does not come in contact with liquids, excessive temperature, or excessive humidity.
- Do not gas sterilize or autoclave the scale.
- Scales contain delicate sensors. Do not bang into doors, walls, etc., as damage may result.

Patient Safety

- This scale is designed for static weighing of patients only. No scale should be used for patient transfer.
- To prevent patient injury, the patient must be attended throughout the entire weighing event.

When operating the scale with batteries:

- If the display indicates low battery, replace the batteries as soon as possible.
- When replacing / inserting batteries, be sure to use all new batteries.

In no event whatsoever shall Pelstar, LLC be liable for damages or injuries arising from or connected with the assembly, use, or misuse of its products.

SPECIFICATIONS FOR THIS SCALE

General

Health o meter® Professional scales use the most sophisticated microprocessor technology available. Each precision instrument is designed to provide accurate, reliable and repeatable weight measurements, as well as other features that make the weighing process simple, fast and convenient.

In order to achieve satisfactory results, the operator must read this manual thoroughly before using this scale.

If any questions arise regarding this scale or its use, please contact Health o meter® Professional Scales Customer Service at 1-800-815-6615.

2210KL / 2210KL4 Scale Specifications

Capacity and Resolution	20,000 grams / 20 kg / 45 lb Resolution: 1 g (to 10,000 g); 5 g (10,001 to 20,000 g); 0.1 oz (0 – 45 lb)	
Power Requirements	6 C-Cell Alkaline batteries	
Environmental	Operating temperatures: 68°F to 85°F (20°C to 30°C) Storage temperatures: 14°F to 122°F (-10°C to 50°C) Maximum Humidity: 95% RH	
Physical Dimensions	2-sided Tray: Width: 24" (610 mm) Depth: 15 3/8" (391 mm) Height: 3" (76 mm)	4-sided Tray Width: 27" (686 mm) Depth: 15 3/8" (391 mm) Height: 3" (76 mm)
	2210KL Product Footprint Width: 25 1/2" (648 mm) Depth: 17 3/8" (441 mm) Height: 5 3/8" (137 mm) Weight: 24.4 lb (11 kg)	2210KL4 Product Footprint Width: 27" (686 mm) Depth: 17 3/8" (441 mm) Height: 5 3/8" (137 mm) Weight: 25.2 lb (11.4 kg)

DEFINITION OF SYMBOLS

Caution or attention required. Risk of potential danger.

Proper positioning of battery polarity.

Degree of protection against electrical shock. Type BF.

Direct electrical current.

CERTIFICATIONS / DISPOSAL

Certification Descriptions

	Pelstar, LLC has been officially certified as the manufacturer of Health o meter® Professional medical devices. The Pelstar, LLC quality assurance system covers the development, production, sales and service of medical scales and measuring systems.
	This product is tested to and meets the requirements of: 1. CSA C22.2 No. 601-1 M90: Medical Electrical Equipment

Scale Disposal

This Health o meter® Professional scale must be disposed of properly as electronic waste. Follow the national, regional or local regulations which apply to you for disposal of electronic waste or batteries. Do not dispose of this device in the domestic waste stream.

ASSEMBLY INSTRUCTIONS

Before Assembly

Each 2210KL / 2210KL4 neonatal/pediatric tray scale is shipped disassembled in one carton. Carefully inspect the carton for shipping damage before unpacking. If damage is found, contact your shipper or a Health o meter® Professional representative immediately at 1-800-815-6615. Claims must be filed with the shipper as soon as possible after receipt of the package. The following information details what you will find inside the main carton as you unpack the parts for assembly.

To prevent scratching any components, carefully remove each assembly from the carton and unwrap the packing materials. Set the carton aside for storage. To avoid damaging the scale parts when unpacking, do not use a box cutter, knife, scissors, or any sharp object to open the protective inner packaging.

Parts List

Carton

- (1) Scale platform
- (1) Acrylic tray
- (6) C-cell Alkaline batteries
- (1) User instructions

Tools Required

#2 Phillips head screwdriver (not included)

ASSEMBLY INSTRUCTIONS (CONT)

The scale is shipped without the required six (6) C-cell batteries installed. The battery compartment is located underneath the scale cover. Use the following procedure to install the batteries:

1. Remove the scale top cover by removing the six (6) screws.


IMPORTANT: When setting the cover aside, do not turn or spin the orientation. The cover only fits properly in one direction and must be replaced in the same orientation as it was removed or the tray will not sit properly on the tray mounting pins.

2. Locate the battery tray (in the rear center of the scale platform). It has a white cover that is held down with two screws. Remove the screws and battery cover.


3. Insert new batteries. Note the orientation that is shown on the bottom of the battery tray.

4. Replace the battery tray cover and screws. Before installing scale cover, press **ON/ZERO** to ensure scale turns on.


5. Reinstall the scale cover. Start partially inserting the (6) screws. Once cover is centered, tighten screws fully. Ensure the scale cover is replaced in the same orientation that it was removed. The cover only fits properly in one direction or the tray will not sit properly on the tray mounting pins. To ensure the cover is placed in the correct direction, the tray mounting pins should be centered in the cover's four circles.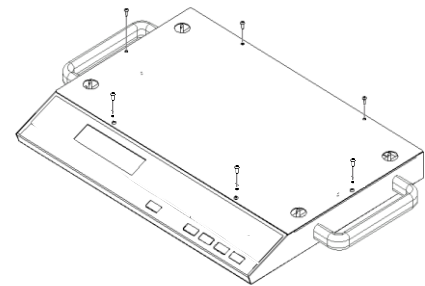

Remove Scale Cover

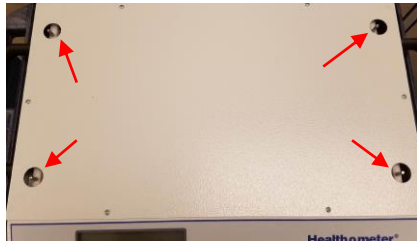
centered

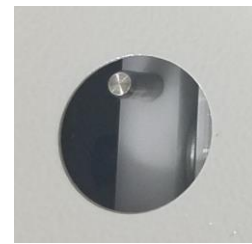not centered

6. Position the scale with the display buttons facing you. The acrylic tray has a built-in measuring tape. Align the tray so that the measuring tape is positioned over the back end of the scale platform.

7. Carefully lower the tray onto the four mounting pins on the scale platform. When the tray is properly aligned with the pins, gently push down on the tray over each pin until it snaps into its locked position. When properly positioned, there will be about a 3/8" space between the platform top and the bottom of the tray.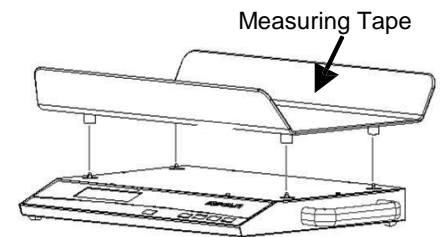

Install Tray

Always lift the scale by the black handles only – never by the tray.

Moving the Scale

The scale should be lifted by the two black handles only.

For your convenience, an optional wheeled cart is available, model # 2210CART.

NOTE: The scale tray should never be used for transporting a baby or objects. The tray surface must be clear of any objects during transport.

SET UP INSTRUCTIONS

Activating/Deactivating Sound

A tone sounds whenever a weight is locked or a button is pressed. (Factory setting is “on”) The scale can be set to stay silent or beep while operating the scale. Follow this procedure to set the preferred option.

1. Press **ON/ZERO** and wait for the scale to display “0.0”.
2. Hold down **MENU** and press **RECALL** . “Set-0” will be displayed and the “0” will be flashing.
3. Press the ▲ button repeatedly until “Set-5” appears. Press **ENTER**.
Note: the ▲ button must be pressed quickly after “Set-0” appears or the scale will return to zero.
4. “11011” will appear on the display. The sound setting is changed with the first digit “11011”.
5. With the first digit flashing on the display, select “1” or “0” for your preferred option. Use the ▲ and ▼ buttons to input your desired setting.
 - a. To keep sounds on, select “1”. Example: Sound on = 11011
 - b. To turn sound off, select “0”. Example: Sound off = 01011
6. Press **ENTER** when your selection is complete. Press **ENTER** to advance to the next digit / setting. If no other settings need to be changed press **ENTER** repeatedly until the scale returns to zero.

Programming Auto-Off Time

The 2210KL / 2210KL4 has been pre-set at the factory to shut down after one minute of inactivity. This can be changed to any increment (from 12 seconds to 25 minutes) to suit your needs.

1. Press **ON/ZERO** and wait for the scale to display “0.0”.
2. Hold down **MENU** and press **RECALL**. “Set-0” will be displayed and the “0” will be flashing.
3. Press the ▲ button repeatedly until “Set-3” appears. Press **ENTER**.
Note: the ▲ button must be pressed quickly after “Set-0” appears or the scale will return to zero.
4. The Auto-Off time currently programmed in the scale will display as minutes and tenths of minutes (i.e. “03.2”); the far left digit will be flashing. Use the ▲ and ▼ buttons to set the desired value, press **ENTER** to advance to the next digit. For example, setting the time to “00.5” will turn the scale off after 30 seconds of no use; “02.0” will turn the scale off after 2 minutes; “25.0” will turn the scale off after 25 minutes.
5. Press **ENTER** when complete to save and return to normal weighing operation.

Unit of Measure Configuration (LB / GM or LB / KG)

The unit of measure may be displayed in lb / grams (i.e. 2541 gram) or lb / kilograms (2.541 kg). (Factory setting is pounds / grams) Follow this procedure to set the preferred option.

1. Press **ON/ZERO** and wait for the scale to display “0.0”.
2. Hold down **MENU** and press **RECALL**. “Set-0” will be displayed and the “0” will be flashing.
3. Press the ▲ button repeatedly until “Set-5” appears. Press **ENTER**.
Note: the ▲ button must be pressed quickly after “Set-0” appears or the scale will return to zero.
4. “11011” will appear on the display. The unit of measure setting is changed with the fifth digit “11011”.
5. Press **ENTER** until the fifth digit is flashing. Use the ▲ and ▼ buttons to input your desired setting.
 - a. To weigh in pounds/kilograms, select “0”.
 - b. To weigh in pounds / grams select “1”.Example: LB / KG display = 11010 LB / GM display = 11011
6. Press **ENTER** when your selection is complete and the scale will return to zero.

SET UP INSTRUCTIONS (CONT)

Unit Lock

The LB / GM button can be disabled so that the scale will weigh in only the selected unit of measure. (Factory setting is “unlocked”) Follow these procedures to set the preferred option.

KG Lock Instructions *(This feature is not available on models with a date code of 0812 or lower.)*

Note: To lock in KG, the scale must be configured LB / KG mode (see pg. 8) and kilograms must be the selected unit on the display before starting the lock procedure.

1. Press **ON/ZERO** and wait for the scale to display “0.0”.
2. Hold down **MENU** and press **RECALL**. “Set-0” will be displayed and the “0” will be flashing.
3. Press the ▲ button repeatedly until “Set-5” appears. Press **ENTER**.
Note: the ▲ button must be pressed quickly after “Set-0” appears or the scale will return to zero.
4. “11010” will appear on the display. The lock setting is changed with the second digit “1010”.
5. Press **ENTER** until the second digit is flashing. Use the ▲ and ▼ buttons to input your desired setting.
To lock into KG select “0”. Example: Locked = 10010
6. Press **ENTER** when your selection is complete. Press **ENTER** to advance to the next digit / setting. If no other settings need to be changed press **ENTER** repeatedly until the scale returns to zero.

GM Lock Instructions

Note: To lock in GM, the scale must be configured LB / GM mode (see pg. 8) and grams must be the selected unit on the display before starting the lock procedure

1. Press **ON/ZERO** and wait for the scale to display “0.0”.
2. Hold down **MENU** and press **RECALL**. “Set-0” will be displayed and the “0” will be flashing.
3. Press the ▲ button repeatedly until “Set-5” appears. Press **ENTER**.
Note: the ▲ button must be pressed quickly after “Set-0” appears or the scale will return to zero.
4. “11011” will appear on the display. The lock setting is changed with the second digit “1011”.
5. Press **ENTER** until the second digit is flashing. Use the ▲ and ▼ buttons to input your desired setting.
To lock into GM select “0”. Example: Locked = 10011
6. Press **ENTER** when your selection is complete. Press **ENTER** to advance to the next digit /setting. If no other settings need to be changed press **ENTER** repeatedly until the scale returns to zero.

LB Lock Instructions

Note: To lock in LB, pounds must be the selected unit on the display before starting the lock procedure.


1. Press **ON/ZERO** and wait for the scale to display “0.0”.
2. Hold down **MENU** and press **RECALL**. “Set-0” will be displayed and the “0” will be flashing.
3. Press the ▲ button repeatedly until “Set-5” appears. Press **ENTER**.
Note: the ▲ button must be pressed quickly after “Set-0” appears or the scale will return to zero.
4. “11011” will appear on the display. The lock setting is changed with the second digit “1011”.
5. Press **ENTER** until the second digit is flashing. Use the ▲ and ▼ buttons to input your desired setting.
To lock into LB select “0”. Locked = 10011
6. Press **ENTER** when your selection is complete. Press **ENTER** to advance to the next digit / setting. If no other settings need to be changed press **ENTER** repeatedly until the scale returns to zero.

SET UP INSTRUCTIONS (CONT)

Unlock Weight Measuring Unit

1. Press **ON/ZERO** and wait for the scale to display "0.0".
2. Hold down **MENU** and press **RECALL**. "Set-0" will be displayed and the "0" will be flashing.
3. Press the ▲ button repeatedly until "Set-5" appears. Press **ENTER**.
Note: the ▲ button must be pressed quickly after "Set-0" appears or the scale will return to zero.
4. "10010" or "10011" will appear on the display. The lock setting is changed with the second digit "10011".
6. Press **ENTER** until the second digit is flashing. Use the ▲ and ▼ buttons to input your desired setting.
To unlock the weight measuring unit select "1". Unlocked = 11011 or 11010
5. Press **ENTER** when your selection is complete. Press **ENTER** to advance to the next digit / setting. If no other settings need to be changed press **ENTER** repeatedly until the scale returns to zero.

OPERATING INSTRUCTIONS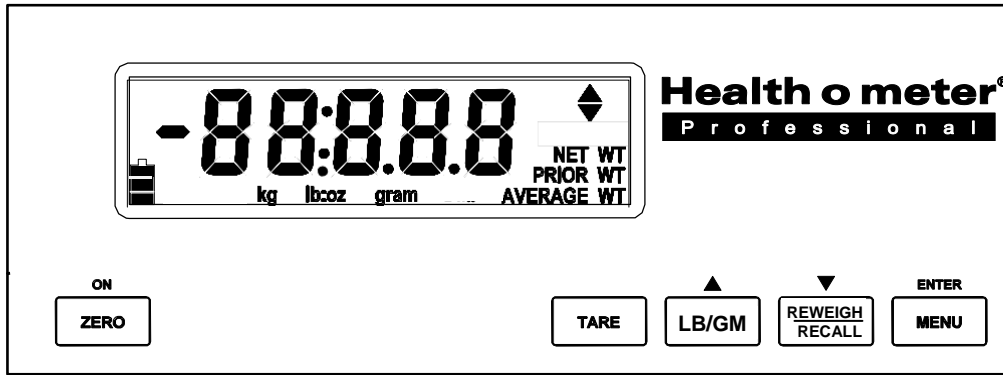

Screen Icons

Icon	Description
	Indicates level of battery power remaining.
kg	Indicates weight mode is in kilograms.
lb:oz	Indicates weight mode is in pounds/ounces.
gram	Indicates weight mode is in grams.
NET WT	Is displayed when a weight is tared.
PRIOR WT	Is displayed after a weight measurement.
AVERAGE WT	Is displayed when motion is detected during a weight measurement.

Keypad Buttons

Note that some buttons perform multiple functions.

Function	Description
ON/ZERO	Press to turn scale on or reset to zero.
TARE	Used to perform the tare function and deducts the weight of an object on the tray.
LB/GM	Toggles the unit of measurement between pounds and grams in LB / GM mode or kilograms and grams in KG / GM mode. See User Setting Options on page 8 for instructions on changing unit of measurement mode. Secondary function is the up arrow and is used for menu selection.
REWEIGH	Allows repeating weighing of the patient without stepping off the scale.
RECALL	Recalls previous weight measurement. Secondary function is the down arrow and is used for menu selection.
MENU	Used for advanced programming functions. Secondary function is the ENTER button and is used for menu selection.

OPERATING INSTRUCTIONS (CONT)

Weighing a Patient


1. Press **ON/ZERO** to power on the scale. If the scale is already on, press **ON/ZERO** to zero the scale. **Note:** **ON/ZERO** should be pressed before each weighing.
2. When “0” appear on the display, place baby on tray.
3. Allow scale to lock onto the weight. The scale will beep and display “PRIOR WT” when the weight is locked.
4. Remove baby from tray. Measurement value remains on the display until it is cleared.
5. To clear the displayed weight and ready the scale for the next weighing, press **ON/ZERO**.

Tare Function

When using this scale, the weight of an object, such as a blanket, on the scale with the baby being weighed can be subtracted from the total weight to determine the net weight of the baby alone. The Tare function automatically performs this subtraction, avoiding the need for manual calculations.

Tare button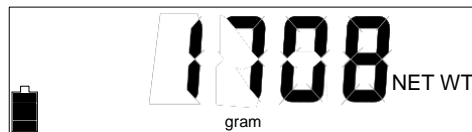


Net weight display

1. Turn the scale on and wait for the display to read “0”.
2. Place the object (i.e. blanket) on the tray. The weight of the object will be displayed.
3. Press **TARE**. “NET WT” and “0.0” will be displayed.
If the object is removed from the platform, its weight will be displayed as a negative value.
4. Place the baby on the tray with the object. The readout will now display the baby’s net weight.
5. Press **ON/ZERO** to return the scale back to normal weighing operation.


Weightlock Function

When using this scale, the weight of any object over 512 grams will be locked on the display even after the baby has been removed from the tray. This allows the caregiver to tend to the needs of the baby first before recording the weight. This value can be recalled even after the scale has turned off.

When the baby’s weight is locked, “PRIOR WT” is displayed. The weight will remain displayed even after the baby is removed from the tray. No further weighings can take place until the weight is cleared by pressing **ON/ZERO** or **REWEIGH/RECALL**.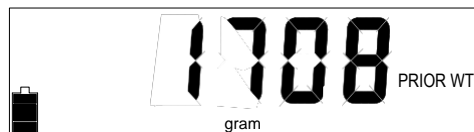

Weightlock display

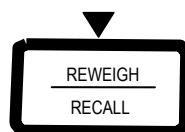Reweigh/Recall button

OPERATING INSTRUCTIONS (CONT)

Reweigh

The Reweigh function is used to reweigh the baby while he or she is still on the tray.

To reweigh, the baby should remain on the tray and a weight measurement should be on the display. Simply press **REWEIGH/ RECALL**.

Recall

The Recall function is used to recall the last weight measurement after the scale has shut down or the display was cleared.

The scale automatically stores the last weight measurement. This value is stored until a new measurement is taken or it is intentionally cleared by pressing **ON/ZERO**.


To recall a weight value after the baby's weight has been measured, the baby has been removed from the tray and the display is off, press **ON/ZERO** and then **REWEIGH/ RECALL**. The stored value will now display.

To clear the recalled weight and unlock the scale for more weighing, either press **RECALL** or **ON/ZERO**.

Motion Detection

Motion detection uses a proprietary technology that automatically provides accurate weight measurement when there is motion on the tray.

If there was considerable motion during the weight measurement, motion detection will activate after seven seconds and "AVERAGE WT" will be displayed along with the weight measurement and "PRIOR WT".

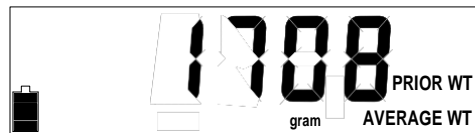Motion detected during weighing

Battery Life Indicators

This scale has excellent battery life with most users getting more than two years from a single set of batteries. Your battery longevity will vary with the frequency of use and the length of the Auto-Off timer. To help ensure you are never without the use of your scale, the display shows the amount of charge left in the batteries.

- **Battery Icon:** Located in the bottom left corner of the LCD display is an icon of a battery. With new batteries, all three segments will be dark. As the batteries become depleted, the segments will turn off.

- **Flashing Battery:** When the battery icon is empty and the frame is flashing, replace the batteries.

NOTE: Properly dispose of old batteries. Install six (6) new alkaline C-cell batteries. Do not mix new and used batteries. See page 7 for battery replacement instructions.

Battery Indicators

CALIBRATION

Your scale has been factory calibrated and does not require calibration prior to use. To check for accuracy, place an accurately known certified weight (3,000 to 5,000 grams recommended) on the platform. If the display reads other than the known weight value by +/- 5 grams, Health o meter Professional recommends calibrating the scale.

Calibration must be done in grams or kilograms using certified weights.

IMPORTANT NOTE: *A certified weight is a weight that has been accurately calibrated and its license is traceable to NBS weights & measures. Weight room free weights are **not** suitable to use as check weights as they may vary from their stated value.*

The 2210KL/2210KL4 is an ultra-accurate scientific instrument. Environmental conditions during the calibration process can cause slight deviations in the calibration results. While these deviations will not affect the overall performance of the scale, we recommend the following procedure for the most accurate calibration results:

1. Press **ON/ZERO** and wait for the display to show "0.0".
2. With no weight on the platform, begin the calibration procedure. **NOTE:** Ensure the scale is on for a minimum of 2 minutes before beginning calibration. If the Auto Off time is programmed for less than 2 minutes, either occasionally tap the scale so it does not power off or change the Auto Off time to longer than 2 minutes.
3. The calibration security code must be entered before attempting to calibrate the scale. Hold down **MENU** and press **RECALL**. Release both buttons. "Set-0" will be displayed and the "0" will be flashing.
4. Press the ▲ button repeatedly until "Set-2" appears. Press **ENTER**. Note: the ▲ button must be pressed quickly after "Set-0" appears or the scale will return to zero.
5. Use the ▲ and ▼ keys to input "125" as the calibration security code. Press **ENTER** to advance to the next digit. After entering the third digit, press **ENTER** and the scale will return to normal weighing mode. The security code needs to be entered only once, unless the scale is powered off. Access to calibration is denied if the proper code is not entered.
6. After "0" appears on the display, wait 5 seconds then press **ON/ZERO** to zero the scale.
7. Place the certified weight onto the scale and wait for the weight measurement to lock. Press **REWEIGH** then quickly hold down **MENU** and press **RECALL**. Release both buttons. **Note:** this button sequence must be done quickly to ensure the weight does not lock after pressing **REWEIGH**.
8. "Set-0" will be displayed and the "0" will be flashing. Press the ▲ button repeatedly until "Set-6" appears. Press **ENTER**.
9. A weight value will display representing the current calculation of the known weight. The grams digit will be flashing. Use the ▲ and ▼ keys to input the proper value of the known weight. Press **ENTER** to advance to the next digit. After entering the weight value, press **ENTER**. "-CAL-" will appear on the display followed by dashes while the scale calibrates.
10. The calibration weight value will appear on the display. Remove weight, wait 5 seconds, then press **ON/ZERO**. Wait for "0.0" to display and verify calibration by placing the same known weight back on the scale. Calibration is now complete. For additional questions or technical support contact Customer Service at 1-800-815-6615.

Calibration Error Code

"Err-4" Scale is below minimum calibration weight. Use at least 3000 grams to calibrate the scale.

TROUBLESHOOTING

Troubleshooting

Before contacting service personnel, refer to the following instructions to check and to correct any failures. If you have further questions, call Health o meter® Professional Scales Customer Service at 1-800-815-6615.

Symptom	Possible Cause	Corrective Action
Scale does not turn on	<ol style="list-style-type: none"> 1. Batteries are not inserted correctly. 2. Batteries are not fully charged. 	<ol style="list-style-type: none"> 1. Check that batteries are “seated” properly in battery tray. See assembly section on page 7 to access the batteries and check their positioning. 2. Check that all new and fresh batteries are properly inserted into the battery tray.
Incorrect weight readings	<ol style="list-style-type: none"> 1. Tray is touching a wall or other object. 2. Tray is not level. 	<ol style="list-style-type: none"> 1. To test, remove all objects from tray. Turn scale on. Display should read “0”. Gently push down on the tray with your finger and release. The display should show some weight value and then quickly return to “0”. 2. Ensure the tray is locked completely onto the four locking pins.
Weight reading consistently too high or too low	<ol style="list-style-type: none"> 1. Scale is out of calibration. 2. Possible broken wire in the connector. 	<ol style="list-style-type: none"> 1. See Calibration section on page 14 to recalibrate the scale. 2. Contact Health o meter Professional Scales Customer Service at 1-800-815-6615.

MAINTENANCE

Cleaning and Disinfecting

Proper care and cleaning is essential to ensure a long life of accurate and effective weighing.

Caution: Disconnect the scale from the AC adapter power source prior to cleaning the unit.

1. Health o meter[®] Professional recommends using one of the following solutions on a soft cloth or disposable wipe:
 - mild soap and water solution
 - 70% isopropyl alcohol
 - solution with 1-5% hydrogen peroxide concentrationAfter cleaning/disinfecting, wipe with a cloth dampened with water and then with a clean dry cloth. To avoid residue build-up or damp surfaces, ensure the screen and scale parts are completely dry after cleaning.
2. Never use rough or abrasive materials to clean the scale, as these will damage the scale's finish.
3. Do not submerge the scale in water or any other liquid.
4. Do not pour or spray fluids directly on the scale.

WARRANTY

Limited Warranty

What does the Warranty Cover?

This Health o meter® Professional scale is warranted from date of purchase against defects of materials or in workmanship for a period of two (2) years. If product fails to function properly, return the product, freight prepaid and properly packed to Pelstar, LLC (see "To Get Warranty Service", below, for instructions). If the manufacturer determines that a defect of material or in workmanship exists, the customer's sole remedy will be replacement of the scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All replaced parts are covered only for the original warranty period.

Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Please save your invoice or receipt. Pelstar dealers or retail stores selling Pelstar products do not have the right to alter, or modify or in any way change the terms and conditions of this warranty.

What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instructions, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover natural disasters, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

To get Warranty Service make sure you keep your sales receipt or document showing proof of purchase. Call (+1) 800-638-3722 or (+1) 708-377-0600 to receive a return authorization (RA) number, which must be included on the return label. Attach your proof of purchase to your defective product along with your name, address, daytime telephone number and description of the problem. Carefully package the product and send with shipping and insurance prepaid to:

Pelstar, LLC

Attention R/A# _____

Return Department

9500 W. 55th St.

McCook, IL 60525

Extended Warranty Available

This scale is eligible for Health o meter® Professional ScaleSurance Extended Warranty Program. ScaleSurance extends the warranty period for an additional two years. This extension to the standard Limited Warranty can be purchased with new scales or for a facility's existing scale before its current warranty has expired. To learn more, visit www.homscales.com/scalesurance/ or contact your medical supply distributor.

PELSTAR, LLC

9500 W. 55th St. • McCook, IL 60525 • USA

1-800-638-3722 or 1-708-377-0600

PLEASE REGISTER YOUR SCALE FOR WARRANTY COVERAGE AT:

www.homscales.com

Health o meter® is a registered trademark of Sunbeam Products, Inc. used under license.

Health o meter® Professional products are manufactured, designed, and owned by Pelstar, LLC.

We reserve the right to improve, enhance, or modify Health o meter® Professional product features or specifications without notice.

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Health o meter® es una marca registrada de Sunbeam Products, Inc. usada bajo licencia.
Los productos Health o meter® Professional son fabricados, diseñados, y de propiedad de Pelstar, LLC.